Ketone Script™

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EASY DIRECTIONS FOR USE:

- Poke a Ballpoint-sized Hole in the Foil Seal.
- Then, Remove a Strip and Immediately Replace the Cap
- Do not touch the Test Area of the Strip
- Dip the Test Pad of the Strip into the Urine Sample or Urine Stream & Remove Immediately
- At 40 Seconds, Compare the Test Pad to the Color Chart on the Bottle Label.
- To Compare Your Result on the Test Strip Color Chart, Fold the Test Pad Backward.
- Ignore Color Changes That Occur After 40 Seconds

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Need Help?

Email Nurse Hatty
Customer Support at:

Hatty@NurseHatty.com

www.NurseHatty.com

NURSE HATTY® KETONE TEST STRIPS

KETONE script™

High-Performance

Reagent strips | Bandelettes De Test
for Urinalysis | De Cétone

150ct.

MADE IN THE USA

Perfect for Dieters

Nurse Hatty®
Caring for You!

Hatty, Inc. Est. 2000

KETONE

Reagent Strips for Urinalysis

TEST your
body Ketone Levels in
Just Seconds

LARGE (160)
16

LARGE (80)
8.0

MODERATE (40)
4.0

SMALL (15)
1.5

TRACE (5)
0.5

NEGATIVE

Manufactured for: Hatty, Inc., 645 Main St. Evanston, WY 82930 USA

URS-1K: Reagent Strips for Urinalysis

DATE
BOTTLE POKED:

USE WITHIN 6 MONTHS OF POKING HOLE IN FOIL SEAL

Expiration Date is for the Shelf Life of Sealed Bottle

A negative result means you don't have any measurable ketones in your urine.

A color change on the test strips means you are in Ketosis (there is detectable ketones in your urine). If you have Ketones in your urine the color on the test pad changes within a few seconds (high-performance) with full results at 40 seconds.